

## INSTRUCTIONS FOR USE


1. Clean the flat surface of the receiver with a dry cloth.
2. Remove one adhesive pad from the strip. (Fig. 1)
3. Press the exposed adhesive surface against the receiver. (Fig. 2)
4. Using a clean tissue with a few drops of rubbing alcohol or use a premoistened alcohol pad clean the skin at the point of contact. Be sure that the skin is free of hair or debris at the point of contact.
5. Remove the remaining paper disc from the pad to expose the adhesive. (Fig. 3)
6. Fasten the receiver by pressing it firmly against the skin for a few seconds.
7. Once the receiver is placed in position it should not be removed unnecessarily as the adhesive quality is reduced each time it is removed/repositioned.

A new adhesive pad should be used daily.



**HAL-HEN**


Garden City Park, NY 11040

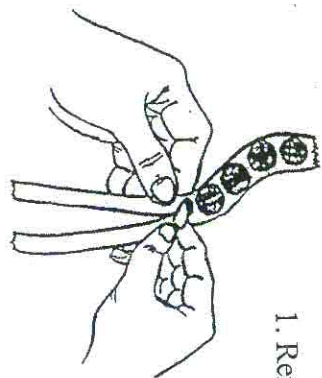

1. Remove pad from roll.



2. Stick to desired area.

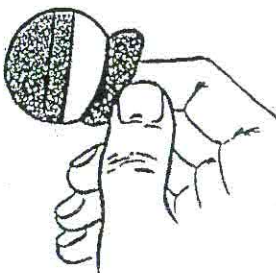

3. Remove backing (band aid fashion).