CA-LX FAQ

Q: What type of lamps will work with the CA-LX?

A: Any household lamp with 100 watt or less will work. But it will not work with lamps with 3-prong plugs

Q: The CA-LX seems to be working and lights up when pressed, but why is my CA-360 not responding?

A:

Most likely, your CA-LX is not connected with your CA-360 yet. Make sure that your CA-360 is connected with the cable provided. Refer to your CA-LX manual for the connecting instructions.

Q: How do I control the plugged-in lamp?

A: Before you plug in the lamp, you must turn its switch to On. After it has been plugged into the CA-LX with the snooze / reset button.