

CA-Mini FAQ

Q: SMS text messages from my cell phone did not activate the Home / Cell Phone Sensor-Transmitter

A1: Confirm that your mobile phone outputs an audible signal on the headset jack.

A2: Select the cell phone mode that provides the strongest and longest vibration time.

A3: Use the short cable to directly connect your mobile phone to the CA-Mini.

Q: Home / Cell Phone Sensor-Transmitter flashes randomly when no calls have been received.

A1: Make sure the unit is not placed on an uneven surface

A2: If it is placed next to a strong RF system, try moving the unit further away.

A3: Press the "RESET" but and replace the cellphone on the platform.

A4: Make sure the unit is not placed on a vibrating surface, for instance a countertop with kitchen tools. Move the unit to a stable surface..

A5: The cell phone was not placed correctly on the resting platform. Carefully replace your cell phone on the holder of the unit.

Q: Will the Home / Cell Phone Sensor-Transmitter detect WEA alerts?

A1: If you cell phone provider provides Wireless Emergency Alerts, the CA-Mini will detect them.

Q: Must I have the Home / Cell Phone Sensor-Transmitter plugged in to the AC Adapter, to use the USB Charging Port to charge my cell phone?

A1: Yes, you must use the AC Adapter.