

ReNew™ Dryer & Freshener User's Guide
(Hearing Aid Dryer & Battery Tester)

Thank you for your purchase of ReNew Dryer & Freshener; please read this user's guide carefully before use and keep it for future reference.

IMPORTANT SAFETY INSTRUCTIONS


WARNING – When using electric appliances, basic precautions should always be followed, including the following:

1. Read all the instructions before using the appliance.
2. To reduce the risk of injury, close supervision is necessary when an appliance is used near children.
3. Do not look directly at the two UV lights located on the front corners of the drying chamber.
4. Only use attachments recommended or sold by the manufacturer.
5. Do not use outdoors. Do not use near water.
6. To disconnect, turn all controls to the off position, then remove plug from outlet.
7. Do not unplug by pulling on cord. To unplug, grasp the plug, not the cord.
8. Unplug from outlet when not in use and before servicing or cleaning.
9. Do not operate any appliance with a damaged cord or plug, or after the appliance malfunctions or is dropped or damaged in any manner. Return appliance to the nearest authorized service facility for examination, repair, or electrical or mechanical adjustment.

5. Remove hearing aids and wipe off any dried wax.

Battery Test:

1. Place a battery against battery tester terminals. Make sure the smaller round battery contact (-) is placed on the gold disc.
2. Press the battery so that its body (+) is pressed against the vertical gold terminal.
3. Battery capacity is indicated by the number of indicators that light up. 4 or 5 indicators means the battery has good capacity remaining. 1 or 2 indicators means the battery has marginal capacity and should be replaced soon.
4. Before testing a new battery, remove battery tab and wait two minutes for the battery voltage to stabilize.

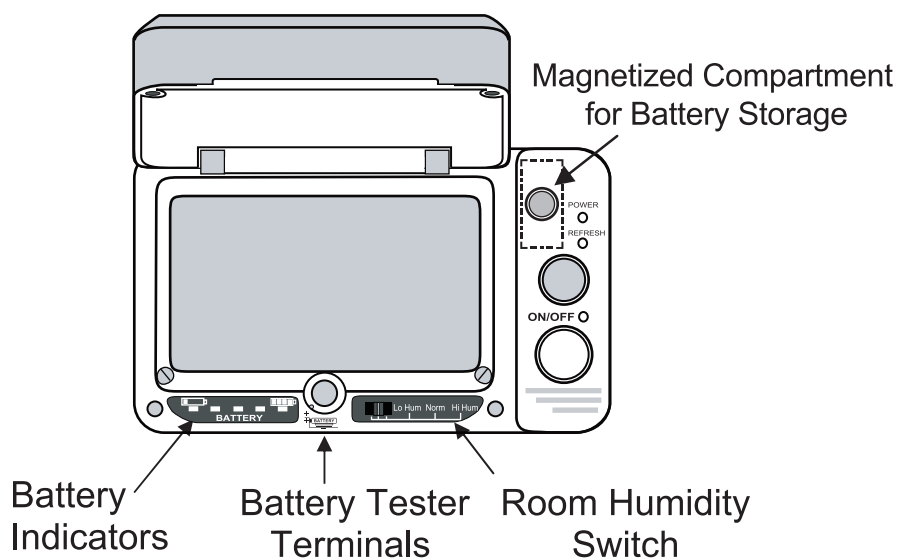

Drying Cycle:

1. Plug AC adapter into the unit and wall outlet. Verify Power indicator is ON.
2. Set Room Humidity switch for the following drying times:
 “Low Humidity” = 2 hrs.
 “Normal Humidity” = 4 hrs.
 “High Humidity” = 8 hrs.
3. Open hearing aid battery cover to expose the battery for drying also. Place hearing aids in the chamber and close the cover.
4. Press ON / OFF button to start the selected drying time. Press the REFRESH button to activate the UV light for one hour (optional)

Caution: Do not look directly at the two UV lights on the front corners of the chamber.

ReNew Specifications @ 25°C ambient:

1	Chamber temp (measured @5 mm above center)	45 +/- 5°C
2	UV light (max on time=1 hr.)	400 nm; < 100mw
3	Drying cycle timer	2 hr, 4hr, 8hr
4	Moisture evaporation rate (tested with water in a 2 mm diameter plastic tube)	Evaporate 18 mm ³ of water after 4 hours of drying at 60% R.H.
5	Battery voltage indicators	5
6	Switching AC adapter input	100 to 240 VAC, 50 / 60 Hz
7	Switching AC adapter output	12VDC, 1000MA
8	Redundant over-temp control	PTC and thermostat
9	Operating temp	0 to 40°C
10	Storage Temp	-10 to 50°C
10	Size (mm)	148x89x51 (5.8"x3.5"x2")
11	Weight (gm)	430 (15 Oz)

Warranty Service Information

ReNew comes with a one-year limited warranty. During this warranty period, from the date of purchase by the original consumer, we warrant ReNew to be free from defects in materials and workmanship under intended home use. In the event that ReNew fails to function properly due to defects in materials or workmanship, within one year of original purchase, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations. Serene Innovations will either repair or replace the unit (with a refurbished unit or a unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase, but within the warranty period, there will be a handling charge of \$9.95 for each return.

This Warranty gives you specific legal rights, and you may also have other legal rights that vary from state to state.

ReNew™ is a trade mark of Serene Innovations, Inc.

Serene Innovations, Inc.

14731 Carmenita Road, Norwalk, CA

Tel: 562 407 5400 www.sereneinnovations.com