



HOW TO USE YOUR **DOCTOR EASY EAR WASHER**

STEP 1. Fill washer bottle with **VERY WARM** [not hot or cold] water as shown by arrow.



Always use filtered, bottled or previously boiled, then cooled, water. To make a cleaning solution, add 1 stick pack of **Earvana™ pH Conditioning Ear Wash Powder** or 2 tsp. white vinegar to very warm water. Attach sprayer to bottle. Turn bottle over a few times to gently mix.

STEP 2. Twist a **DISPOSABLE TIP** onto the end of the nozzle. Your ear washer comes with 3 tips. Additional tips are available and fit ALL Doctor Easy Ear Washers.

STEP 3. If you have an **EAR BASIN**, place it under the ear to catch fluid or lean over sink. Rapidly pump solution into the ear to flush out wax. **For heavy build up, this can take vigorous washing. You may need to refill the bottle.**

READ BEFORE WASHING:

Do **NOT** use if you have ear tubes, pain, or bloody discharge.

EAR WASHING SHOULD NOT HURT! Pressure is normal, but if you feel pain, **STOP** and consult a physician.

EAR WASHING MAY CAUSE *DIZZINESS*. *USE CAUTION.* USE VERY WARM WATER, NOT *HOT* OR *COLD* WATER.

ALWAYS USE ON A PERSON WHO IS SEATED.

DISCARD TIP AFTER USE. Do **NOT** re-use tips.

ALLOW EAR WASHER TO DRY BEFORE STORING IT. See storage bag for cautions and cleaning suggestions.

SEE DOCTOR EASY EAR WASHERS IN USE! View our Product Videos at www.Doctor-Easy.com; WaxRx.com, Or YouTube for "Elephant Ear Washer", "Rhino Ear Washer," or our newest home-use kit, "WaxRx".

**SEE REVERSE OF THIS CARD FOR
AWESOME EAR WASHER ACCESSORIES** →