
# EZ-OFF™


Please review instructions before use.


#### **ATTACH**

- 1. Lay mask on a flat surface "face side" up. Place the EZ-OFF inside the mask string with the raised middle section facing up and the hook facing down.
- 2. Lift the hook and slide the mask string under it.
- 3. Slide the string into the top slit as far as it will go.
- 4. Slide the string into the bottom slit as far as it will go, then center EZ-OFF on mask string.

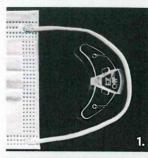



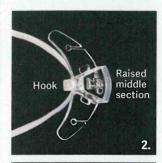

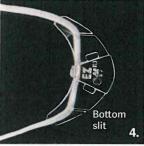



# WEAR Grasp the raised middle section of EZ-OFF with your thumb and forefinger, pull around one ear at a time. Be sure the raised middle section is closest to your head and the hook faces

your ear.



## REMOVE

To remove grasp the raised middle section pull backwards from behind your ear, then away from your head.



### **HEARING AIDS**

Put your hearing aid(s) on first. Be sure to position the top part of the EZ-OFF between the outer ear and hearing aid. If necessary use a mirror.

FOR HOW TO VIDEOS VISIT EZEE-OFF.COM