egger LP/H

egger LP/H Xpress

Modellierpasten
Modeling pastes

Allgemeine Produktinformation

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General product information

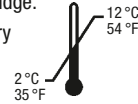
The egger LP modeling pastes can be used for numerous applications:

- Repair of BTE and ITE earmolds made of UV polymerizates or other hard acrylates
- Reducing or sealing of ventings
- Post-production attachment of retention points
- Application of a color marking
- Manufacturing and modification of ventings in ITE earmolds

With the modeling paste Xpress you can manufacture ventings, execute repairs or modifications in the same high quality while saving up to 50 % time. The modeling paste Xpress was especially adapted to the wavelength coverage of the light pistol Megalux CS (REF 51550 ff) to reach an optimum polymerization time. The handy tubes or cartridges guarantee efficient and clean processing.

Stability / Batch: See crimped end of tube or see cartridge.

Storage: The material should be stored in a cool and dry place away from food stuff. Always keep the tube or cartridge closed. Protect from light.

Notes:

- Always close tube or cartridge immediately after use.
- Avoid direct skin contact with the liquid material! The hands should be protected by wearing appropriate gloves (see material safety data sheet) before working with the material.

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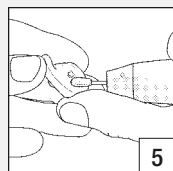
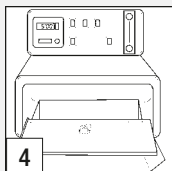
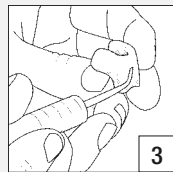
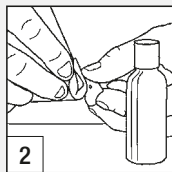
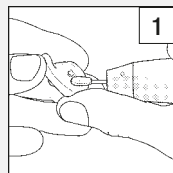
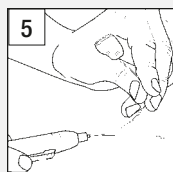
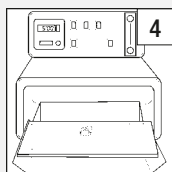
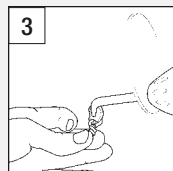
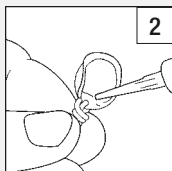
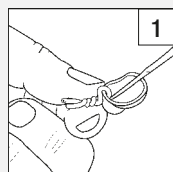
Ventings in ITE shells

Processing

The processing can be done in one of the following egger EL UV-curing units:

egger EL 1	REF 51100
egger EL 1 plus N ₂	REF 51110
egger EL 2	REF 51200
egger EL 3	REF 51300
Light pistol	REF 51550

Please refer to the operating instructions of the used unit and the below listed polymerization times.

Processing note

Processing at a room temperature of approx. 23 °C (73.4 °F) ± 2 °C (35.6 °F) and a relative air humidity of 50 % ± 5 %.

Polymerization times

egger EL UV-curing unit:	Polymerization time:
EL 1, EL 1 plus N ₂ , EL 2, EL 3	2–6 minutes
Light pistol	20–180 seconds

Modifications / Repairs

1. First of all, the part of the otoplastic that is to be processed has always to be roughened with Corundum paper, grit 180 (REF 40704).
2. Then, clean and degrease the surface carefully with Surface Cleaner (REF 31600 ff).
3. Apply the modeling paste directly out of the tube or cartridge or with a modeling instrument (REF 31215) and even the transitions to the ground.
4. Then, depending on the coating thickness applied, let it cure for about 2 to 6 minutes in the UV-curing unit or for about 20 to 180 seconds under the light pistol.
5. After the polymerization time, wipe off the surface of the earmold thoroughly with a cloth soaked with Surface Cleaner (REF 31600 ff).
6. If necessary rework the earmold with a cutter and prepare it for lacquering.

Ventings in ITE shells

1. Roughen the inner surface of the ITE shell with a matching round bur (REF 70100 ff). Then place the vent tube with wire (REF 92100 ff) in the ITE shell and twist the wires on the outside. Press the vent tube with wire to the inner wall of the shell with a modeling instrument (REF 31215).
2. Cover the vent tube (REF 92100 ff) with modeling paste beige (REF 31000) or modeling paste Xpress (REF 31020).
3. Then, depending on the coating thickness applied, let it cure for about 20 to 180 seconds under the light pistol (REF 51550). Remove the vent tube (REF 92100 ff) carefully after polymerization.
4. Post-curing of the ITE shell with the created venting in the LP cleaner bath or under gas (with the EL 1 plus N₂) for 4–6 minutes in the UV-curing unit.
5. Then glue the face-plate with Uniglu (REF 39501), mark the vent outlet opening, using a vent ear light (REF 26611 ff) and open it with a matching drill.

Counter indications:

Do not use if hypersensitive to acrylate/methacrylate.

Please follow the product information and the material safety data sheet.