## egger LP/H

klar, rosa-transparent, beige, flesh, rot, blau

## egger LP/HS

klar, rosa-transparent

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## General product information

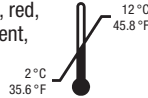
LP/H material can be processed in one of the following egger EL devices:  
egger EL 1 REF 51100 egger EL 2 REF 51200  
egger EL 1 plus N<sub>2</sub> REF 51110  
Always observe the operating instructions of the device you are using and the polymerization times indicated below.

**Delivery units:** See catalog


**Expiry date and charge:** See bottom of the bottle or tube crimp.

**Storage:** The material should be stored in a cool and dry place, away from foodstuff. Keep container always closed. Protect from light.

For LP/H beige, flesh, red,  
blue; LP/HS transparent,  
pink-transparent

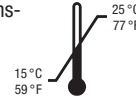For LP/H trans-  
parent, pink-  
transparent

### Notes:

- Close the container after use.
- In case of air bubbles in the material heat up the open container (but covered with the lid) to a temperature of approx. 50 °C (122 °F) for 15 to 30 minutes.
- Avoid direct skin contact with the liquid material! Hands should be protected with appropriate gloves before working with the material and finishing the blanks.

**Notes for processing:** Before working with the material, it must have a temperature of approximately 23 °C (73.4 °F) at a relative air humidity of 50% ± 5%.

## Processing

### Manufacturing of hard BTE earmolds or ITE shells

- Fill the LP/H or LP/HS material into the negative mold made of duplicating material (REF 40900 ff). Make sure that the negative mold is completely filled and bubble free.
- Cover it with black foil (REF 53200 or REF 53100).
- Put the filled negative mold into the UV-polymerization unit and let it polymerize according to the following polymerization times. Gas flow is **not** required if you work with the UV-polymerization unit EL 1 plus N<sub>2</sub>.

LP/H and LP/HS transparent and pink-transparent	
EL 1	2 min. 30 sec. to 3 min.
EL 1 plus N <sub>2</sub>	2 min. 30 sec. to 3 min. (without gas)
EL 2	3 min. to 3 min. 30 sec.

LP/H beige	
EL 1	2 min. 30 sec. to 3 min.
EL 1 plus N <sub>2</sub>	2 min. 30 sec. to 3 min. (without gas)
EL 2	3 min. to 3 min. 30 sec.

LP/H flesh	
EL 1	1 min. 10 sec. to 1 min. 20 sec.
EL 1 plus N <sub>2</sub>	1 min. 10 sec. to 1 min. 20 sec. (without gas)
EL 2	1 min. 20 sec. to 1 min. 30 sec.

LP/H red and blue	
EL 1	30 sec.
EL 1 plus N <sub>2</sub>	30 sec. (without gas)
EL 2	1 min. to 1 min. 10 sec.

### ATTENTION:

**The mentioned polymerization times are guide values, they may vary depending on the size of the earmold and the transparency of the negative mold.**

- Remove the black foil after polymerization.
- Pour the liquid material into a container that is impervious to light. **This material can be reused.**
- After having completely poured out the liquid material, put the negative mold and the blank but without the black foil into the UV-polymerization unit again. **We recommend placing ITE shells in the unit with the opening facing downward.** Gas flow is **not** required if you work with the UV-polymerization unit EL 1 plus N<sub>2</sub>.

LP/H and LP/HS transparent and pink-transparent	
EL 1	1 min. to 2 min.
EL 1 plus N <sub>2</sub>	1 min. to 2 min. (without gas)
EL 2	2 min. to 3 min.

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- Take the blank out of the negative mold and put it into a transparent container (REF 50107) filled with LP-Cleaner (REF 31300 ff). Then put it into a UV-polymerization unit for the second polymerization. **If you work under gas, put the blank into a transparent box with cover (REF 51114).**

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- Then remove the blank. If you worked with LP-Cleaner (REF 31300 ff) rinse the blank under running water.

### Treatment and lacquering:

- The blank can now be shaped with the special egger carbide cutters.
- Before lacquering the surface, roughen it with abrasive paper, grit 180, (REF 40704) then clean and degrease it with Surface Cleaner (REF 31600 ff). Afterwards you can coat the earmold/ITE shell with the LP/H lacquer (REF 30600 ff), LP/H lacquer antibac (REF 30621 ff) or LP/W lacquer (REF 30700 ff).