


Getting Started

1. Download the **FireflyPro Mobile** app from the app store.

Supports iPad Mini 2 and iPhone 6 or newer.

2. **Charge Otoscope before first use.**

The indicator light will turn green when battery is fully charged.

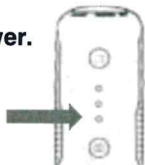

3. Turn Otoscope power **ON** by pressing the on/off button



4. On your mobile device, go to your WiFi settings and connect to your Firefly Device.

Name: FF_Camera_xxxx

Default Password: 12345678

5. Launch the **FireflyPro Mobile** app.

Quick Tips

1. Turn silver metallic roller left and right to focus the image.
2. Ensure specula is inserted completely and the viewing circle is centered on the screen.
3. If the app displays the message "Firefly Device not Detected" Close the app and reconnect your device in WiFi settings.