


# Quick start installation guide for an induction loop


please refer to product user guide for complete user instructions

## 1 system components

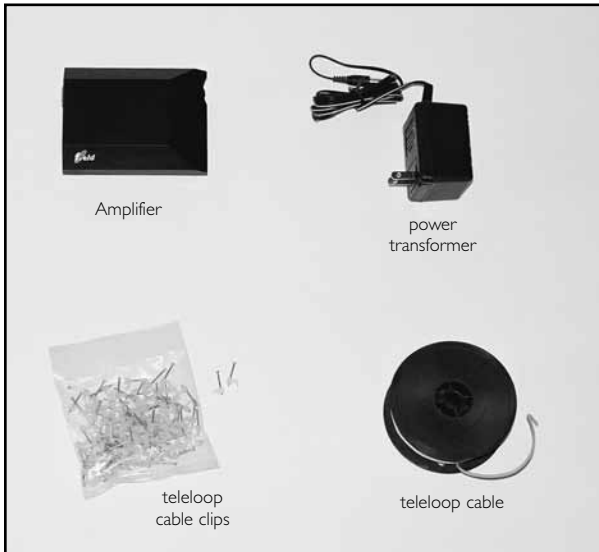

Package may vary depending on product and model

## 2 installation tools

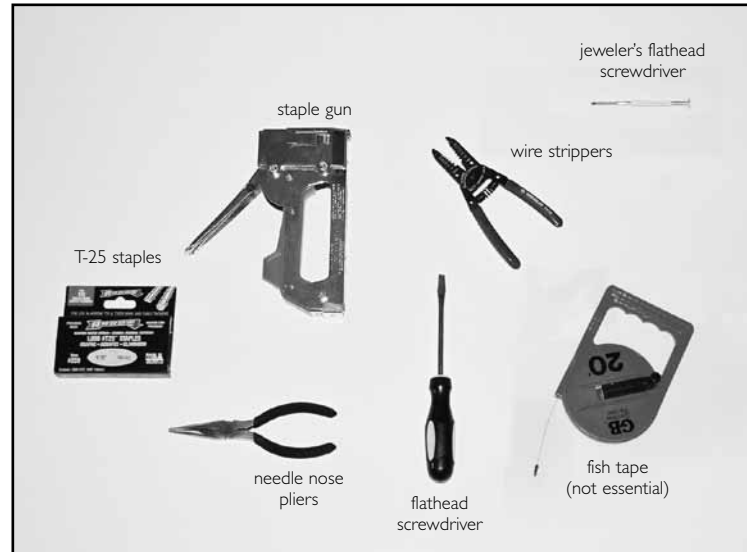

Gather these tools before you begin installation

## 3 Pull up a section of the carpet along the baseboard with the needlenose pliers




## 4 Insert a flathead screwdriver to lift the carpet away from the baseboard



## 5 Guide the fishtape under the carpet toward the other side of the room



## 6 Pull the fishtape up and double knot the teleloop cable through the open slot

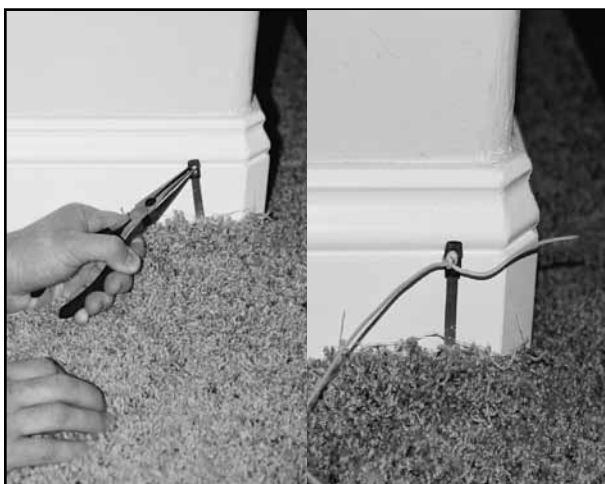

## 7 Now pull the attached teleloop cable back under the carpet to the other side



## 8 Untie the knotted teleloop cable from the fishtape



## 9 Stretch the cable along the baseboard and tuck it in with the screwdriver



## 10 Tack in the cable clips or use a staple gun to secure points along the way



## 11 Around doorways without carpet, staple the cable at the bottom and pull taut




Continued on the back side

12 When the room has been completely looped, strip the end of the teleloop cable




13 Remove the connector from the amplifier, insert cable ends, and tighten screws

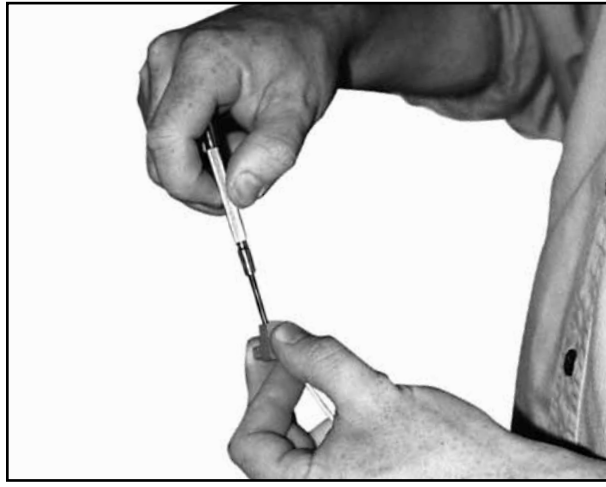


14 Replace the connector of the cable loop into the back of the amplifier unit



15 Plug one end of the cable into the **Radio/TV** connection on the amplifier

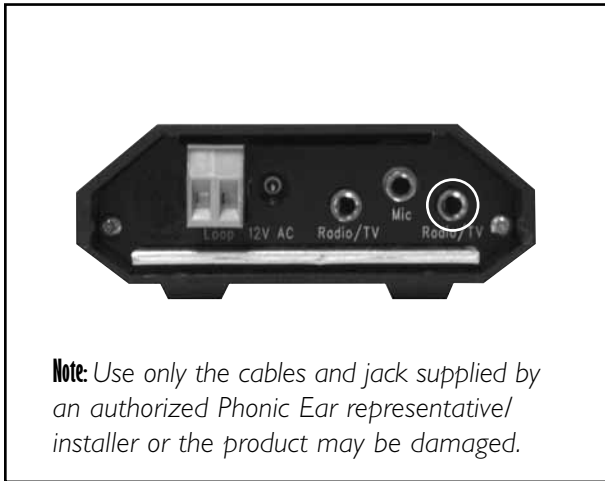

16 Plug the other end of the cable into the **Line Out** connection on the TV

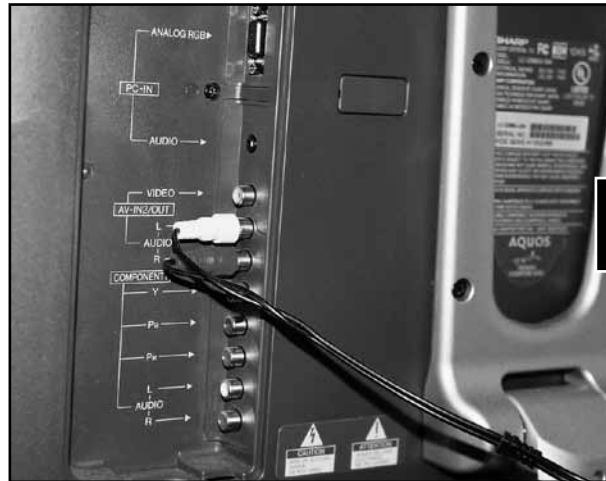


17 Plug the microphone cable into the **Mic** connection on the back of the amplifier



If the sound source cannot be attached to the teleloop using the Line Out connection, place the microphone in front of the television or stereo speaker.

18 Plug the transformer into a wall socket and into the power connection on the amplifier

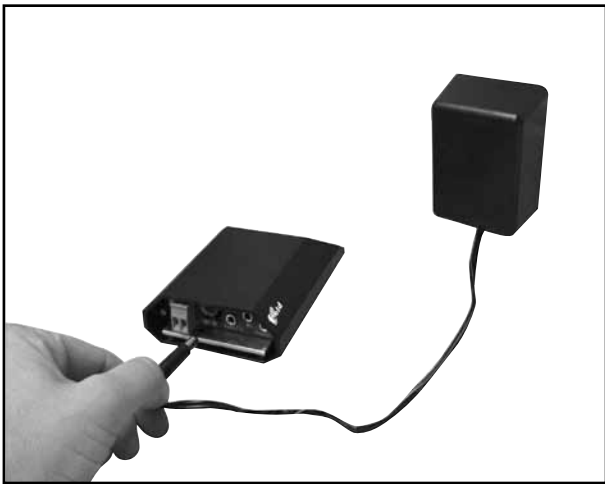

Typical an indication light on the front of the loop eluminates should go on. Adjust the output strength on the front of your loop to the desired level.

19 Flip the amplifier over to adjust the bass and treble if available.



20 Finally, simply set the user's hearing aid to the "T" position when using the teleloop

