## Complete Home Alerting Solution

### HA360MVS

## Main Alerting Unit with Bed Shaker Smoke / CO Detection

Copyright © 2015 Sonic Alert. All right reserved.

# TABLE OF CONTENTS

Important Safety Instructions.....	1
Product Description .....	4
HomeAware™ Features .....	4
Package Contents .....	5
Identification Features – Front.....	6
Identification Features – Back .....	7
Setting Up Your Main Unit for the First Time .....	8
Getting Started / Time and Alarm Settings... ..	9
Setting Alarm 1 .....	9
Setting Alarm 2.....	9
Turning Off Alarm .....	10
Snooze and Alarm Duration .....	11
Flash, Audio and Vibrator Alarm Settings.....	12
Alarm 1 – Setting Flash Alarm Options.....	12
Alarm 1 – Setting Audio Alarm Options .....	12
Alarm 1 – Setting Vibrator Alarm Options.....	13
Alarm 2 – Setting Alarm Options .....	13
Test Mode - Testing Flash, Audio, Vibrator and Lamp .....	13
Select 12 or 24 Hour Mode.....	14
Speaker Volume and Tone Settings .....	14
Auto Dimmer and Alert Settings .....	15
Language Settings and Removing a Linked Device .....	16
Battery Back Up .....	17
USB Charging .....	18
Vibrator / Bed Shaker .....	19
Smoke / CO Alert .....	19
Telephone Alert for Landline Phones .....	20
Optional Accessories.....	20
Regulatory Compliance .....	21
General Data .....	22
Warranty.....	23

## **IMPORTANT SAFETY INSTRUCTIONS**

When using your HomeAware™ HA360MVSBS, basic safety precautions should always be followed to reduce the risk of fire, electric shock or injury to persons including the following:

1. Read and understand all instructions, if there is something you do not understand please call our customer service.
2. Do not use any electrical product near a bathtub, wash basin, kitchen sink or laundry tub, in a wet basement, or close to a swimming pool or anywhere else there is water.
3. Do not use a telephone anywhere near a gas leak.
4. Avoid using a telephone (other than a cordless type) during a storm. There may be a risk of electrical shock from lightning and electrical storms.
5. Always unplug this product from the wall power outlet before cleaning. Do not use liquid cleaners or aerosol cleaners on the product. Use a damp cloth for cleaning only.
6. Always place this product on a stable surface. Serious damage and/or injury may result if the product falls.
7. Do not cover slots, vents or openings on this product. This product should never be placed near or over a radiator or heat register. This product should not be placed in a built-in installation unless proper ventilation is provided.
8. Always operate this product using electrical voltage as specified on the base unit or in the user guide. If you are not sure of the voltage in your home, consult your dealer or local power company.
9. Do not place anything on the power cord. Install the product where no one will step or trip on the cord.
10. Never push any objects through the slots in the product. They can touch dangerous voltage points or short out parts that could result in a risk of fire or electrical shock. Never spill liquid of any kind on the product
11. Do not overload wall outlets or extension cords as this can increase the risk of fire or electrical shock.

## **IMPORTANT SAFETY INSTRUCTIONS Continued**

12. Unplug this product from the wall outlets and refer servicing to the manufacturer under the following conditions:
  - A. When power cord or plug is damaged or frayed.
  - B. If water or liquid has been spilled into or on the product.
  - C. If the product has been exposed to rain or water.
  - D. If the product does not operate normally by following the operating instructions. Adjust only those controls that are covered by the operating instructions. Improper adjustment may require extensive work by a qualified technician.
  - E. If the product has been dropped or if the case has been damaged in anyway.
13. To reduce the risk of electrical shock, do not take this product apart. Opening or removing covers may expose you to dangerous voltages or other risks. Incorrect reassembly can cause electric shock when the appliance is subsequently used.
14. Never install telephone wiring during a lightning storm.
15. Never install telephone jacks in damp or wet areas.
16. Never touch uninsulated telephone wires or terminals unless the telephone line and power supply have first been disconnected.
17. Always use caution when installing or modifying telephone lines.
18. Use only the power cord and batteries indicated in this manual. Do not dispose of batteries in a fire. They may explode. Check with local codes for possible special disposal instructions.
19. This product should not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on or near the product.
20. Batteries shall not be exposed to excessive heat, such as sunshine or fire or any other heat source.
21. The Main unit's power supply plug is used as the disconnect device, which shall remain readily operable.

## **PRODUCT DESCRIPTION**

Sonic Alert's HomeAware™ Complete Home Alerting Solution is the first of its kind to incorporate supervised safety, security and peace of mind. Sonic Alert has developed industry leading technology with its proprietary TruAlert™. TruAlert™ D-RF networking provides the latest in wireless technology and is on guard constantly, searching for any problems 24/7. These potential problems will be reported on its large, bright red display and in easy to see text.

With TruAlert™ networking you'll feel safe and secure knowing your system is monitoring itself every 2-3 minutes 24/7 even in the event of a power failure.

HomeAware™ with TruAlert™ networking will alert you to a landline telephone, cell phone, video phone, and the doorbell ringing. HomeAware™ comes complete with a doorbell transmitter, powerful bed shaker, extra loud dual alarm with up to 105 decibels, an extra bright flashing xenon strobe and a convenient USB cell phone charger.


## **HOMEAWARE™ FEATURES**

- TruAlert™ D-RF safe & secure 24/7 networking technology.
- Water resistant doorbell transmitter with extended range.
- Powerful Sonic Alert bed shaker.
- Scrolling caller ID and alert messages are shown on its bright red, large 2" display.
- Extra bright strobe flasher.
- Dual alarm clock with alarm and snooze duration, auto dimming, scrolling display.
- Alarm with up to 105 decibels of loud sound.
- USB outlet for Smartphone charging.
- Telephone alert for landline phones, faxes and video phones, RJ11 interface.
- Test mode to test alarm functions.
- Smart extended battery backup, up to 72 hours of backup with regular AA alkaline type.


# PACKAGE CONTENTS

1. HomeAware™ – Main Unit
4. Vibrator/Bed Shaker
5. Telephone Line Cord
- ..6 Power Supply Adapter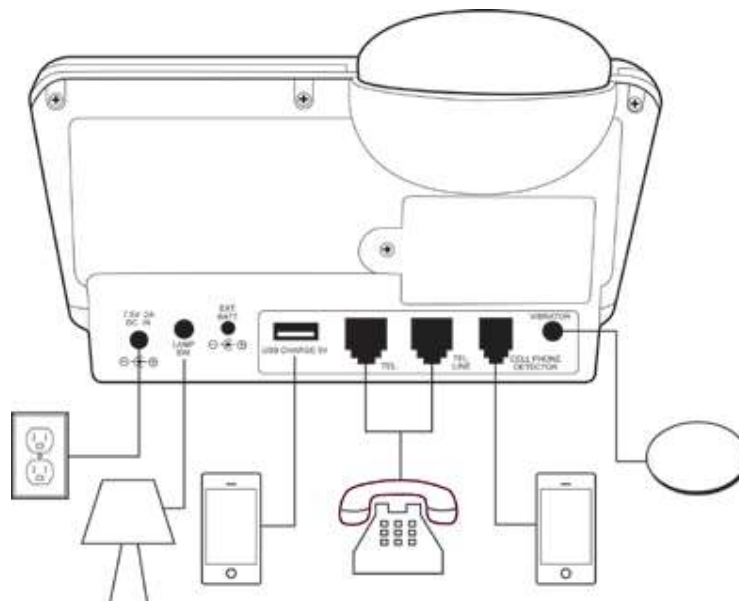


## IDENTIFICATION FEATURES - FRONT

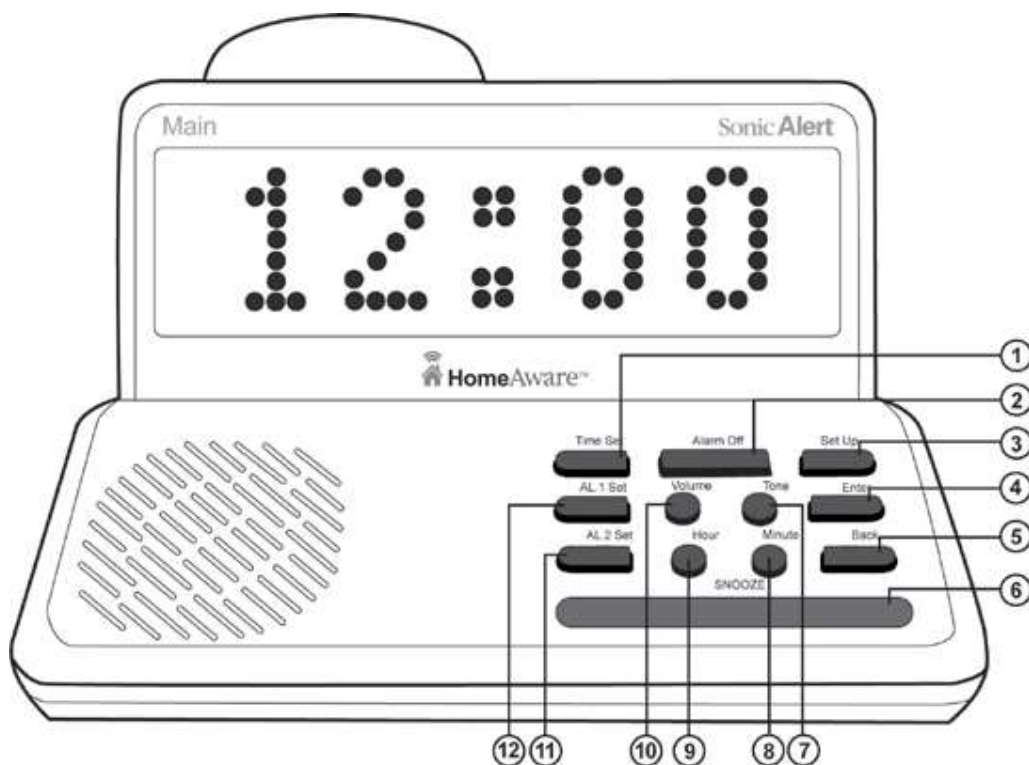1. Time Set
2. Alarm Off
3. Set Up
4. Enter
5. Back
6. SNOOZE
7. Tone
8. Minute
9. Hour
10. Volume
11. AL.2 Set
12. AL.1 Set

## IDENTIFICATION FEATURES – BACK

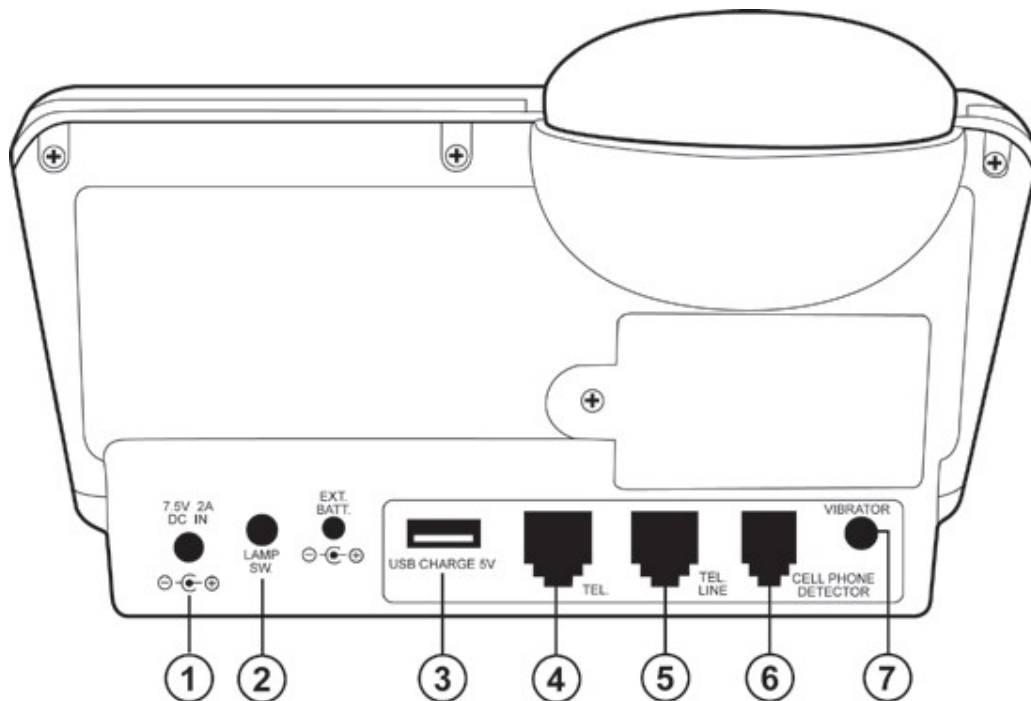1. POWER SUPPLY INPUT 7.5V 2A DC IN
2. LAMP SWITCH / BRIDGE UNIT (Optional)
3. USB CHARGE 5V
4. TEL.
5. TEL. LINE
6. CELL PHONE DETECTOR (Optional)
7. VIBRATOR/BED SHAKER


# SETTING UP YOUR MAIN UNIT FOR THE FIRST TIME

## What Is Included With Your HomeAware™:

1. HomeAware™ – Main Unit
2. Doorbell Transmitter
3. Doorbell Transmitter Chime Box Connector Cable
4. Vibrator/Bed Shaker
5. Telephone Line Cord

## Out of Box First Time Use

1. Remove all contents of the box. Make sure items 1 through 5 are all available, as listed above.
2. Plug in the power supply (see diagram below 7.5V 2A DC IN). Observe the correct color coding on the back of the unit and on the back of the power supply. Note: If using a remote receiver the color for the Main unit and remote receiver unit will be different colors.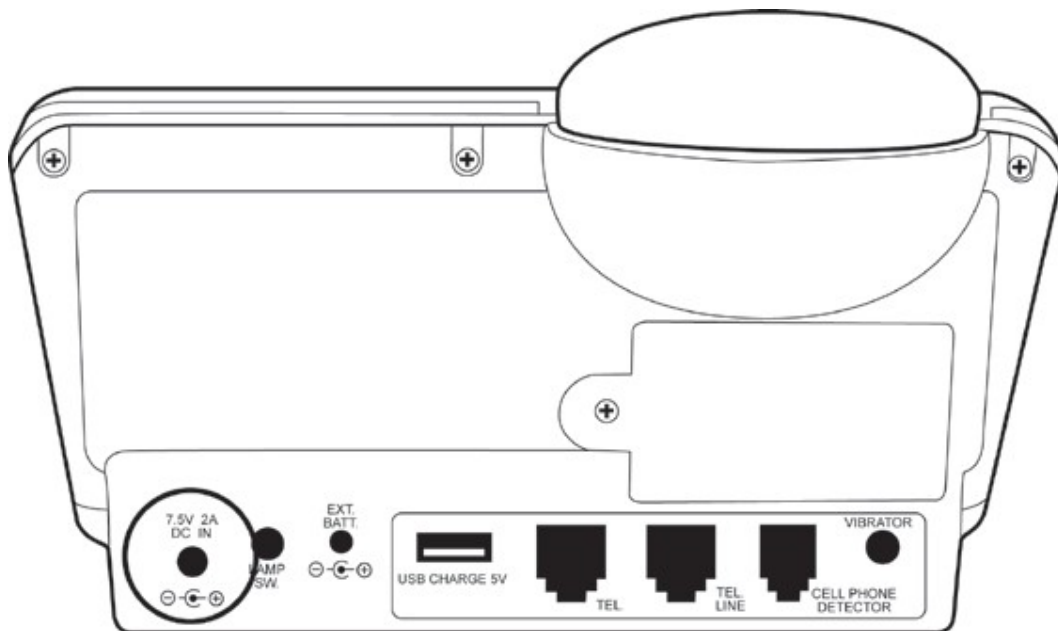

# GETTING STARTED

## TIME AND ALARM SETTINGS

### Setting the Time of Day

1. Press the **Time Set** button. Blinking red dots shown on the display indicate time set mode has been activated.
2. Press the **Hour** buttons up or down and the **Minute** buttons up or down to set desired time. The PM indicator is located on the top left hand side (see diagram next page).
3. Press the **Enter** button when done to complete time setting.
  - Display dots will stop blinking to indicate time has been set.

### Setting Alarm Times - there are two alarms AL.1 and AL.2.

#### Setting Alarm 1

1. Press and hold **AL.1 Set** button. Blinking red dots shown on the display indicate alarm set mode has been activated.
2. Press the **Hour** buttons up or down and the arrow **Minute** buttons to set desired alarm time.
3. Press the **Enter** button when done to complete, a red LED light will appear on the display's left side indicating alarm 1 is now set (see diagram next page).


#### Setting Alarm 2

Follow the same steps for AL.1 above to set AL.2. AL.2 will have its own indicator light also located on the left as A2.

# TIME AND ALARM SETTINGS Continued

## Turning Off Alarm

To turn off alarm AL.1 or AL.2 press **Alarm Off** or **SNOOZE** button (see setting up **SNOOZE** following page).

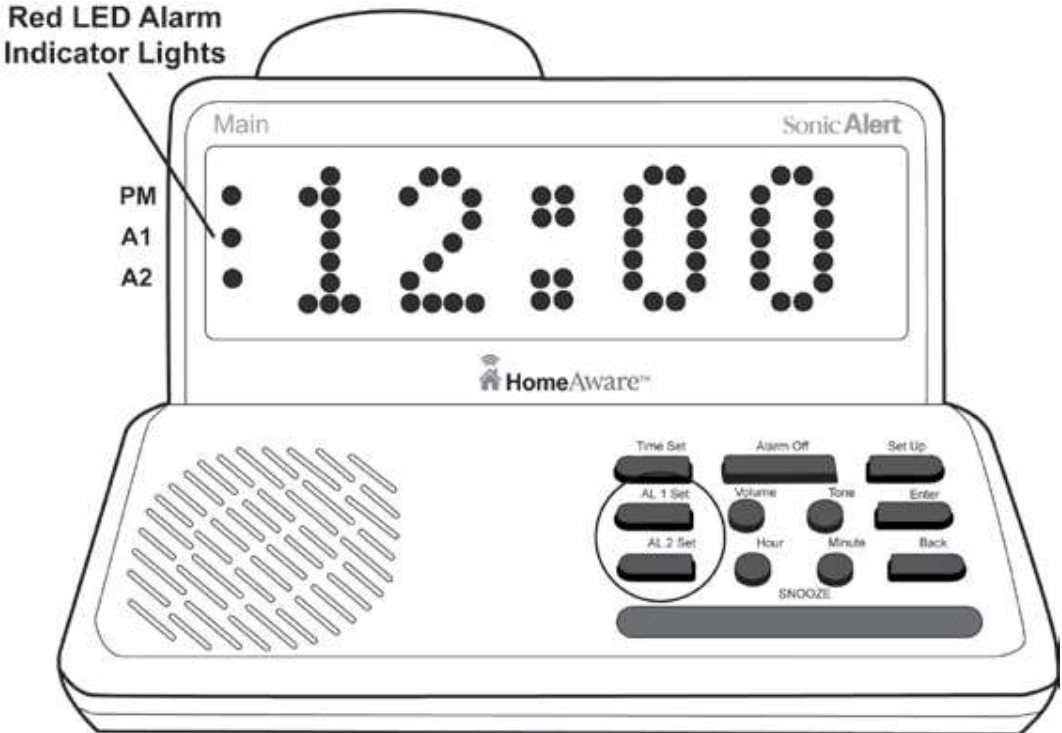# SNOOZE AND ALARM DURATION

## Setting Alarm Duration

Alarm duration will allow you to select the length of time that the alarm will run (5 min to 30 min).

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display, press **Enter**, the word **ONTIME** will be shown on the display, press **Enter**.
3. Press the **Tone** button to set the desired alarm duration (5 min – 30 min) and press **Enter** to complete alarm duration setting.

## Setting Snooze Duration

Snooze duration will allow you to select the length of time that the snooze alarm will run (5 min to 30 min).

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display, press **Enter**, the word **ONTIME** will be shown on the display
3. Press the **Tone** button until the word **SNOOZE** is displayed, press **Enter**. The default snooze duration is set for 10 minutes if this is your desired snooze duration press **Enter**. If you would like a shorter or longer snooze duration (5 min to 30 min) press the **Tone** button to toggle until desired snooze duration is displayed and press **Enter**.

# FLASH, AUDIO AND VIBRATOR ALARM SETTINGS

## Alarm1 – Setting Flash Alarm Options

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display, press **Enter**.
3. Press the **Tone** button until the word **ALARM1** is shown on the display.
4. Press the **Enter** button and the word **FLASH** will be shown on the display.
5. Press the **Enter** button and the word **ON** will be shown on the display.
6. Press the **Tone** button to toggle between **ON** or **OFF**, press **Enter** on your selection.

## Alarm1 – Setting Audio Alarm Options

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display, press **Enter**.
3. Press the **Tone** button until the word **ALARM1** is shown on the display.
4. Press the **Enter** button and the word **FLASH** will be shown on the display.
5. Press the **Tone** button and the word **AUDIO** will be shown on the display, press **Enter**.
6. Press the **Tone** button to toggle between **ON** or **OFF**, press **Enter** on your selection.

## FLASH, AUDIO AND VIBRATOR ALARM SETTINGS Continued

### Alarm1 – Setting Vibrator Alarm Options

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display, press **Enter**.
3. Press the **Tone** button until the word **ALARM1** is shown on the display.
4. Press the **Enter** button and the word **FLASH** will be shown on the display.
5. Press the **Tone** button until the word **VIBR** is shown on the display, press **Enter**.
6. Press the **Tone** button to toggle between **ON** or **OFF**, press **Enter** on your selection.
7. While in the Set Up mode, the alarm indicators on the right side will light up showing the alert options chosen (sound, flash, and vib).

### Alarm2 – Setting Alarm Options

Follow the same steps above when setting **ALARM2**.

### Test Mode (Testing Flash, Audio, Vibrator and Lamp)

1. Press **Enter** and **Back** buttons together, the word **TEST** will be shown on the display. The audio, vibrator, strobe and lamp flash will be activated, so you will be assured that all three alerts are working.
2. Press and hold the **SNOOZE** button for 2 seconds to complete test mode.

## 12 OR 24 HOUR MODE AND VOLUME / TONE SETTINGS

### Select 12 or 24 Hour Mode

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display, press **Enter**.
3. Press the **Tone** button until **12 / 24** is shown on the display, press **Enter**.
4. Press the **Tone** button to toggle between **12 HR** and **24 HR**, press **Enter** on your selection.

### Speaker Volume and Tone Setting

1. Press the **Volume** up and the **Hour** down buttons to select the preferred volume level. There are 8 volume levels and an OFF level to completely turn off the sound.
2. Press the **Tone** up and the **Minute** down buttons to select the preferred tone level. There are 15 different tones to select from.

## AUTO DIMMER AND ALERT SETTINGS

### Automatic Display Dimmer

The HomeAware™ has a built in auto dimmer. It will automatically dim when the room gets darker. The auto dimmer can also be turned off and will stay bright even when the room gets dark.

### Turning Off the Automatic Dimmer

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display.
3. Press the **Tone** button until the word **DIM** is shown on the display, press **Enter**.
4. Press the **Tone** button to toggle between **ON** or **OFF**, press **Enter** on your selection.

### Alerts Option (Number of Alerts)

The HomeAware™ has alert options that will let the user pick how many times they will be alerted 3, 6 or 9 times.

### Setting the Alerts Option

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button until the word **ALERTS** is shown on the display, press **Enter**.
3. Press the **Tone** button to toggle between **3**, **6**, or **9**, press **Enter** on your selection.

# LANGUAGE SETTINGS AND REMOVING A LINKED DEVICE

## Language

The HomeAware™ can be set to display three different languages, English, Spanish or French.

## Setting Language Preference

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display.
3. Press the **Tone** button until the word **LANG** is shown on the display, press **Enter**.
4. Press the **Tone** button to toggle between **ENG**, **ESP** or **FRA**, press **Enter** on your selection.

## Removing a Linked Device


When a linked device needs to be removed (replacing an existing device with a new one or a device has been lost)

1. Press **Set Up** button and the word **LINK** will be shown on the display.
2. Press the **Tone** button and the word **CLOCK** will be shown on the display.
3. Press the **Tone** button until the word **REMOVE** is shown on the display, press **Enter**.
4. Press the **Tone** button to toggle between devices, press **Enter** on your selection.

# BACKUP BATTERY

## Battery Back Up

1. To set up battery backup unplug the Main unit from the 120v power supply.
2. Install 5 new AA alkaline batteries in battery compartment located on the bottom of the Main unit.
3. The LED display will be dimmed when running on the battery backup only. Upon any alert such as doorbell or telephone the display will be at its maximum brightness. After any alert the display will go dim again to help conserve battery power.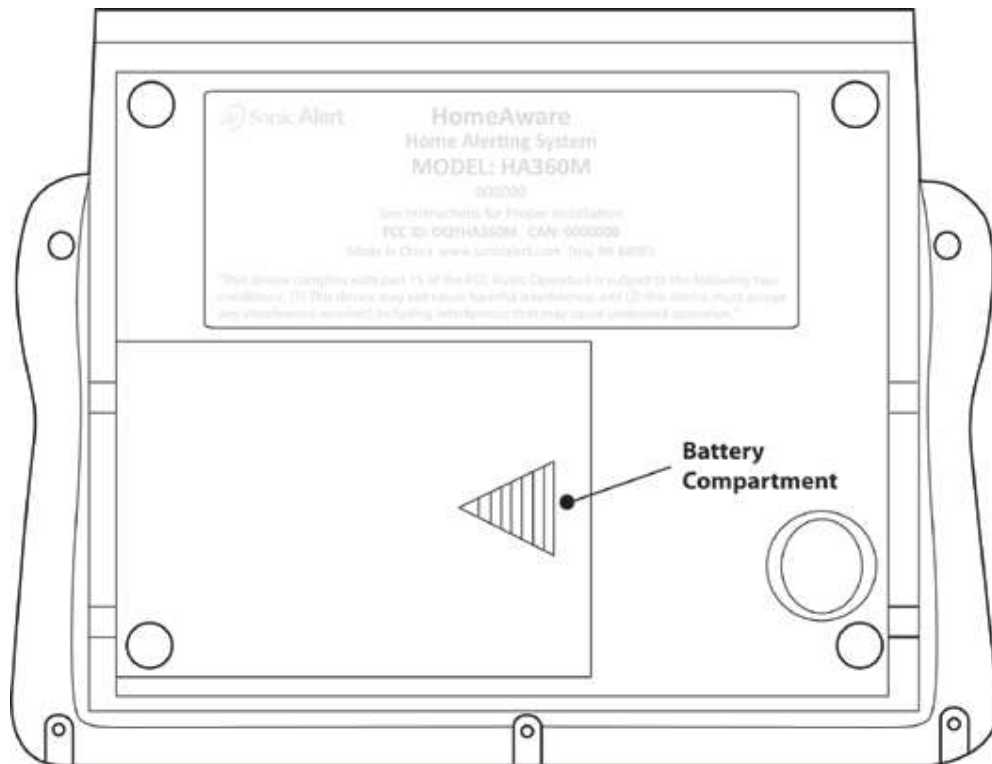


# USB CHARGING

## USB Charging

The HomeAware™ is equipped with a convenient USB charging port on the back panel that allows you to recharge many brands of mobile phones, portable music players, and other handheld devices that are supplied with USB charging cables, tablets not included.

To charge your device, connect a USB charging cable that was supplied with your device to the USB charging port on the back of the Main unit. Observe the charging indicator on your device to confirm that it is charging. Note: Charging time will vary depending on the current that your device requires. Therefore some devices will take longer to charge than others.

\*The HomeAware™ USB charger handles 5 volt devices only. It will not charge an iPad® or tablet.

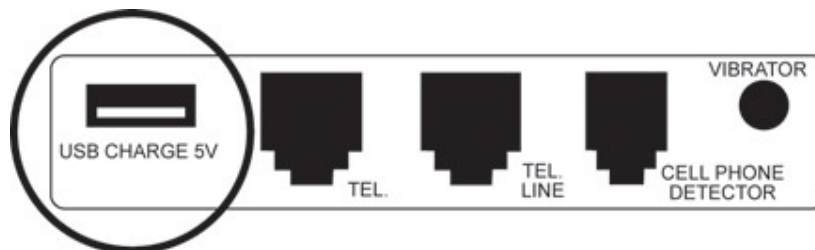## **VIBRATOR / BED SHAKER**

### **Vibrator/Bed Shaker**

Your HomeAware™ comes with a powerful vibrator/bed shaker. This powerful vibrator/bed shaker is most effective when placed between the mattress and box spring or under a pillow. This feature is a great way to wake up as it requires no loud sound that can wake others. This bed shaker can be programmed to shake for the alarm clock alert, or for all alerts including baby cry, doorbell, smoke / CO, and telephone alerts.

To setup the vibrator/ bed shaker simply plug the vibrator into the back of the HomeAware™ where it says vibrator.

## **SMOKE / CO ALERT**

The HomeAware™ will alert you to the sounds produced by the smoke and CO detectors already installed in your home.

### **Detecting the Different Alert Sounds**

The Main Unit detects loud sounds and will alert you to the source of that sound. If it detects a loud sound from a smoke detector that meets the T3 standard used for SMOKE DETECTORS, “SMOKE” will scroll on the display of the HomeAware™ Main Unit.

If it detects a loud sound from a CARBON MONOXIDE (CO) DETECTOR that meets the T4 standard, “CO” will scroll on the displayed of the HomeAware™ Main Unit.

### **For All Other Sounds**

If a loud sound is detected that does NOT match the T3 or T4 standard, then “ALERT” will scroll on the HomeAware™ Main unit in a loud, repeating pattern.

## TELEPHONE ALERT FOR LANDLINE PHONES

The HomeAware™ has the capability to alert you to a standard land line telephone. Follow the steps below to install a telephone to the HomeAware™.

1. Place the HomeAware™ Main unit close to a RJ11 standard telephone wall jack. If the provided telephone line cord is not long enough to reach the HomeAware™ you will need to either move the HomeAware™ closer to the input jack or purchase a longer line cord.
2. Plug the phone line cord into the RJ11 wall jack.
3. Plug the opposite end of the phone line cord into the back of the HomeAware™ where it says **TEL LINE**. You are now ready to receive alerts for all phone calls coming in from this phone.
4. You can also utilize the HomeAware™ to add a phone to its input jack on the back of the unit. Simply plug your phone into the back where it says **TEL** your landline phone is now ready to make and receive calls.

## OPTIONAL ACCESSORIES

- HomeAware™ extended range Deluxe Remote Receiver's provide notification with a powerful xenon strobe flasher, and the first of its kind scrolling text display.
- HomeAware™ Basic Receiver is a more cost effective solution for those on a budget.
- Smoke and CO (carbon monoxide) Detection Transmitters.
- Baby Cry monitoring Transmitter.
- Smartphone calls and personal paging of all push alerts from HomeAware™. Your smartphone application will provide alerts from your HomeAware™ system right on your smartphone (Baby, Smoke, Call, Door, etc.).
- SonicBridge™ allows the HomeAware™ to link with any traditional Sonic Alert remote receivers, while also providing for a personal lamp flasher.

# **REGULATORY COMPLIANCE**

## **PART 68 OF FCC RULES INFORMATION**

The HA360MVSBSB complies with Part 68 of FCC Rules. You must, upon request, provide the FCC registration number and REN to your local phone company. Both numbers are shown on the HA360MVSBSB's rating label.

## **PART 15 OF FCC RULES INFORMATION**

The HA360MMVSBSB complies with part 15 of the FCC rules, and with RSS-210 of Industry Canada. Operation is subject to the following two conditions; (1) This device might not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide responsible protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. However, there is no guarantee that interference will not interfere in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- i. Reorient or relocate the receiving antenna.
- ii. Increase the separation between the equipment and receiver.
- iii. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- iv. Consult the dealer or an experienced radio /TV technician for help.

Modification not expressly approved by the manufacturer and could void the user's authority to operate the equipment under FCC rules.

## GENERAL DATA

UL listed Power Adaptor: AC100-240V~, 50/60Hz  
Power Consumption: Adaptor Output: DC7.5V 2A

 <p>The Lightning flash and arrowhead within the triangle is a warning sign alerting you of dangerous voltage inside the product.</p>	<p><b>CAUTION:</b> TO REDUCE THE RISK OF ELECTRICAL SHOCK, DO NOT REMOVE THE COVER (OR BACK) FROM THE PRODUCT. NO USER SERVICEABLE PARTS INSIDE. REFER SERVICING TO YOUR SONIC ALERT DEALER.</p>	 <p>The exclamation mark within the triangle is a warning sign alerting you of important instructions accompanying the product.</p>
--	--	--

**WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.**

The normal function of the product may be disturbed by Strong Electro Magnetic Interference. If so, simply reset the product to resume normal operation by following the instruction manual. In case the function could not resume, please use the product in other location.

## GENERAL DATA Continued

- Attention should be drawn to the environmental aspects of battery disposal.

- This indicating separate collection for electrical and electronic equipment.

### SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The HomeAware™ HA360MVSBS is warranted against manufacturing defects in materials and workmanship for one (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the HomeAware™ HA360MVSBS without charge for parts and labor.

Simply contact our customer service department to obtain an RA (return authorization number) and instructions on how to process your product for return. Call 1-888-864-2446 or e-mail to [RMA@sonicalert.info](mailto:RMA@sonicalert.info). You'll need a copy of your receipt to process the return authorization.

**Sonic Alert Warranty Center**  
**1081 West Innovation Dr. Kearney, MO 64060, USA.**

**Made in China**  
**Sonic Alert, Troy, MI 48083**  
**www.SonicAlert.com 1-888-864-2446**

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Copyright © 2015 Sonic Alert. Todos los derechos reservados.