



Clarity HA40 Handset Amplifier

The HA40 is a battery-operated device that uses a 9V battery (included in the packaging). Most issues related to static or the device not working is associated with the need to replace the battery. The frequency in which the battery needs to be replaced depends on how often the amplifier is used throughout the day as well as how much amplification is applied with each use; the higher the amplification setting, the quicker the battery drain. On average, the 9V battery will last about 4-6 months although in some cases, replacing the battery every 2 months may be normal.

BATTERY INDICATOR

The battery indicator light (see figure below) illuminates every time the handset is lifted. When the battery indicator light is on, it indicates that the battery level is okay. If the indicator light does NOT come on when the handset is lifted, the battery needs to be replaced.



When the battery is drained, if the volume level is moved to the lowest (down) position, the handset will work. When the volume level is at any other position, the handset WILL NOT operate.

As the battery is approaching its end life, it is possible for the battery indicator light to illuminate, however, the quality of the incoming caller's voice may be intermittent, low in volume, and/or contain a lot of static.

FOR AN AC-POWERED OPTION WHERE BATTERIES ARE NOT USED, SEE [CS-WIL95](#)

www.oaktreeproducts.com

800.347.1960

otp@oaktreeproducts.com