
**SERENE®**  
**PROFESSIONAL**  
**Hearing Aid**  
**Vacuum Cleaner**  
**Operating Guide**

Model  
**HC-200**


**FEATURE IDENTIFICATIONS:**

The HC-200 hearing aid cleaner comes with two different size suction tips for compatibility with different hearing aids.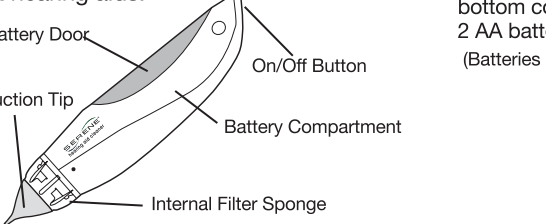

**INSTALL FOUR AA ALKALINE BATTERIES AS SHOWN:**

Place 2 AA batteries in the bottom compartment and 2 AA batteries on top. (Batteries not included)

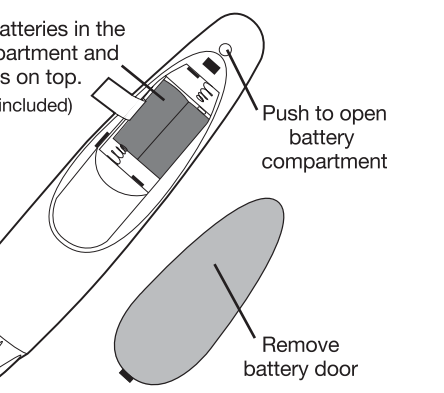English

**USING YOUR HC-200 :**

**WARNING---DO NOT use the HC-200 on your ears. The HC-200 is for cleaning your hearing aids only. You MUST first remove your hearing aids from your ears.** If necessary, consult your hearing healthcare professional on how best to use the HC-200 on your hearing aids.

**Hearing Aid Canal Cleaning**

- Insert the tapered tip gently into the hearing aid's canal and start vacuuming, as shown. Do not insert too deep into the hearing aid canal or press with excessive force as this may cause damage. (Figure A.)


English

**Surface cleaning**


- With the HC-200 turned off, remove the tapered tip and replace it with the square tip, as shown. Brush gently the outside surface of your hearing aid with the small cleaning brush.
- Use the square suction tip to vacuum the outside surface of your hearing aid as shown. (Figure B.)

**Cleaning and Maintenance Instructions**

- Filter and Suction Tip--- Unscrew the suction tip from the hearing aid vacuum cleaner to remove the filter and empty any water collected. If necessary, wash the filter and suction tip under running water, dry thoroughly, and reinstall. (Figure C.)
- Hand Pump--- Only wipe clean with a dry or damp soft cloth. Do not use solvents and do not wash in dishwasher or submerge in water.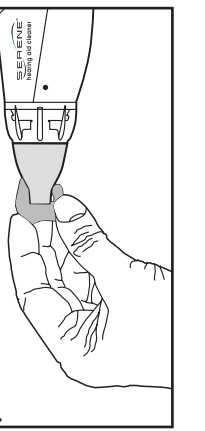

English

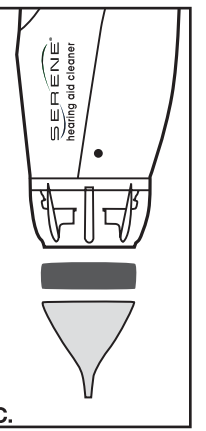**WARRANTY:**

Your HC-200 (except batteries) comes with a one year limited warranty. We warrant during this warranty period, from the date of purchase, to the original consumer, the HC-200 to be free from defects in materials and workmanship under intended home use.

In the event that the HC-200 fails to function properly within 30 days of purchase, return it to place of purchase for exchange.

If failure occurs within one year of original purchase due to defects in materials or workmanship, return the unit, with a check

in the amount of US \$9.95 (shipping charge) and proof of purchase (sales receipt) or packing slip, no exceptions to:

**Serene Group Inc.**  
 14731 Carmenita Road. Norwalk, CA. 90650

We will either repair or replace the unit (with a refurbished unit or unit of equal condition) at no cost to you.

This warranty does not apply to any product that has been damaged due to accident, abuse, misuse, neglect, or if used on

English

**WARRANTY (CONT.):**

electrical frequency or voltage other than marked on product and/or described in manual. Any defects or errors caused by unauthorized alterations, repairs, and/or tampering are also not covered by this warranty.

If a defect covered by this warranty should occur, promptly contact a Customer Care Representative by phone to obtain a Return Authorization Number (RAN) and shipping instructions before shipping the product to us. Any shipment without a RAN will not be accepted and will be returned to you at your

expense. Any authorized returned product must be accompanied with proof of purchase and a brief explanation of the problem. For out of warranty repair and service, please contact our Customer Service Care:

**SERENE®**  
 Serene Group , Inc.  
 14731 Carmenita Road, Norwalk, CA 90650  
 Toll Free: 866.376.9271 Local: 562.407.5400  
 (8AM-5PM PST, M-F)  
 www.sereneinnovations.com

Copyright © 2018 Serene Group , Inc. All rights reserved.

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Copyright © 2018 Serene Group, Inc. All rights reserved.
Serene Group, Inc.