Super

LARGE WIPE

WARNING KEEP OUT OF REACH OF CHILDREN

See outer carton for additional precautionary statement.

Sani-Cloth®

GERMICIDAL DISPOSABLE WIPE

ACTIVE INGREDIENTS:

n-A	Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl _N ylbenzyl ammonium chlorides	
eth	ylbenzyl ammonium chlorides	0.25%
n-A	Álkyl (60% C.,, 30% C.,, 5% C.,, 5% C.,)	
dir	Álkyl (60% C ₁₄ , 30% C ₁₄ , 5% C ₁₂ , 5% C ₁₈) nethyl benzyl ammonium chlorides	0.25%
Iso	propyl Alcohol	55.00%
	HER INGREDIENTS	
TO	TAL	
Do	es not include the weight of the wipe	

Directions for use It is a violation of Federal law to use this product in a manner inconsistent with its labeling. Tear open packet. Unfold wipe and use as directed. Discard after single use. TO DISINFECT AND DEODORIZE: To disinfect nonfood contact surfaces only: Unfold a clean wipe and thoroughly wet surface. Allow treated surface to remain wet for a full two (2) minutes. For heavily soiled surfaces, use a wipe to pre-clean prior to disinfecting. Let air dry. These directions also apply to Mycobacterium bovis BCG (Tuberculosis) at 68°F (20°C).

CONTAINS: 1 LARGE WIPE 5 x 8 in (12.7 x 20.3 cm) Net Wt. 0.18 oz. (5.1 g)

PRECAUTIONARY STATEMENTS Hazards to Humans and Domestic Animals WARNING. Causes substantial but temporary eye damage. Do not get in eyes or on clothing. Avoid contact with skin. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using restroom. Remove and wash contaminated clothing before reuse.

FIRST AID Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes.

NOT A SKIN WIPE FOR USE ON HARD NON POROUS SURFACES ONLY

EPA REG. NO.: 9480-4 EPA EST. NO.: 9480-NY-1

Manufactured by: Professional Disposables International, Inc. Orangeburg, NY 10962-1376 ◆ USA 1-800-999-6423



