

Perfect Dry[®]

Electric Drying System for Hearing Aids

Initially we want to congratulate you to have chosen the Drying System PerfectDry[®].

Your Hearing Aids undergo each day multiple aggressions (perspiration, change of temperature, etc.), which often causes a premature oxidation of the electronics, malfunctions, even breakdowns and thus for you, a deprivation of your Aids during several days. With the PerfectDry[®] System, you will decrease considerably all these constraints.

Use

The best moment to dry your Hearing Aids is in the evening before sleeping.

Open the Battery Houses completely, place your Hearing Aid with molds in PerfectDry®, then close it again.

Connect then the Jack with the Box (Fig. 1), and then connect the Adaptor to a socket. Use the switch on the back of the Perfect Dry® system to start the drying cycle (Fig. 2). The green pilot light comes on (Fig3).


The System is equipped with a timer (approx. 2.15 hours). Beyond this time the PerfectDry® stops automatically (the Green Indicator is switched off). This avoids you having to disconnect the System once finished drying. When you want to restart or stop the drying cycle, use the switch (Fig.2).

During certain period of the year (ie. Summer) or if you work in a very wet place (ie. Professional Kitchen), you can optimize drying by adding a Drying Capsule in the Box (Fig.4). Ask advice to your Audiologist.

Perfect Dry® Cleaning


Clean at least once a week the interior of the Box using a dry and clean Tissue.

Fig.1

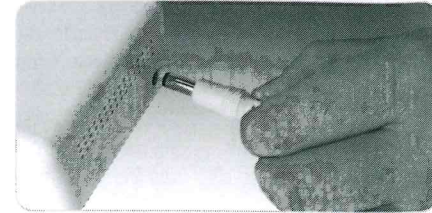


Connect the Jack in the Perfect Dry® delicately

Fig.2 - Back

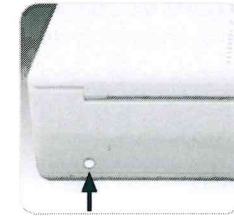

Switch to start drying cycle

Fig.3 - Front

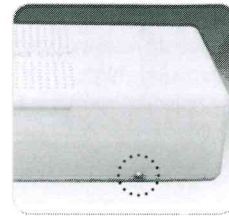

Green pilot light on = drying cycle activated

Fig.4



The Drying Capsule will optimize the operation of the Perfect Dry®

Warning

Use only for the drying of your Hearing Aids.

Keep it in a dry and clean place.

Do not use in water.

Do not cover.

Do not use chemicals to clean it.

Never open the electric part of the system.

In case of opening, the Guarantee would be cancelled automatically.

Technical Description

Use only the Adaptor delivered with the PerfectDry®.

The PerfectDry® operates on a temperature of 38° to 42° C.

Guarantee

PerfectDry® is guaranteed 1 year as from the date of purchase, date of invoice being valid.

This Guarantee is valid within the framework of a use conform to the regulations of use.

In the event of a breakdown, disconnect it from the sector and contact your Audiologist. From this Guarantee are excluded deteriorations due to a foreign cause, a misuse, a non conform connection, the effects of a lightning or of an electric overpressure.

Information

This Product is subject to the European Directive 2002/96/CE concerning the prevention and the management of the electric and electronic components scraps.

The Symbol below indicates that this Product is subject to a selective collection.

We recommend that you put in contact with your Audiologist or our Aftersales Service to follow these Directives and to exclude all possible environmental damages.



 MG Développement
ZA aéroport de Montpellier
F-34470 Pérols