

Conversor Pro

Quick Start Guide



1. Using the Conversor Pro with your hearing aid

Switch your hearing aid to "T" setting or Telecoil mode.

See your hearing instrument user guide or consult your audiologist



2. Microphone set up

Press the ON/OFF switch to turn the microphone on

Switch to boost the sound level

Switch for zoom and omni-directional modes



3. Receiver set up

Hang the receiver around your neck

Press ON/OFF switch to turn the receiver on

Set the volume adjustment buttons to provide you with a comfortable sound level



4. Using the microphone

The microphone can be held by the user and pointed at the sound source or the person talking; placed on the table for group discussions; or given to a speaker, teacher or lecturer for use in a classroom, lecture hall or church

5. Using in directional or zoom mode

Point the microphone directly at the person talking




or

6. Using in omni-directional mode

Place the microphone on the table in the middle of the group to hear everyone talking



How to charge Conversor Pro

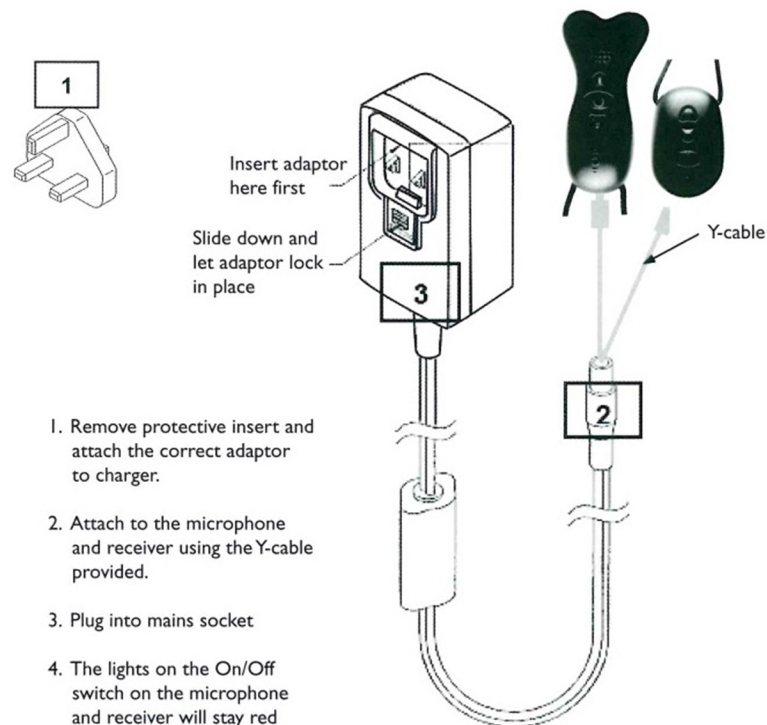

1. Remove protective insert and attach the correct adaptor to charger.
2. Attach to the microphone and receiver using the Y-cable provided.
3. Plug into mains socket
4. The lights on the On/Off switch on the microphone and receiver will stay red until the unit is fully charged when they will turn green.

Charger time on fully discharged batteries	Approximate battery life
25 minutes	1 hour
40 minutes	2 hours
65 minutes	4 hours
90 minutes	6 hours
150 minutes	8 hours