

Troubleshooting Guide:

PerfectDry Lux Rechargeable

*The indicator lights are reflective of the hearing aids. These can be found at the bottom of the front of the unit.

* If (2) hearing aids in use, then both lights will illuminate. If only (1) hearing aid in use, then only a single light will illuminate.

GREEN FLASHING LIGHT = Charging and drying process is complete.

The unit will always run a 60 minute drying cycle, regardless of the actual charging time needed

GREEN SOLID LIGHT = Charging cycle is complete and 100% battery. The drying process is only program running.

NO LIGHTS = Unit is off, finished charging, drying, and disinfecting

ORANGE FLASHING LIGHT = ZPower unit is not being detected. Check the connection from the ZPower cable to the PerfectDry Lux

ORANGE SOLID LIGHT = There is a problem with dryer. This would be a defective unit and need warranty replacement

RED SOLID LIGHT – There is a problem with the ZPower Charger. This would be a defective ZPower charger



Temperature controlled circa 30°C.
Positive influence on charging capacity.



UV-C Cleaning



Drying through Heat
and Ventilator



LED information on
charging and drying

