

DM50 Impression Gun Instructions:

1. To insert the plunger, hold down the release button and insert plunger through the side on the gun with the larger opening
2. While holding down the release button, insert plunger flat side first into the gun and slide all the way through until it is secure
3. Remove the impression cartridge cap and put on mixing tip
4. Double check the plunger to make sure it is fully inserted in the gun
5. Lift the cartridge support piece on the top
6. Install cartridge into the hole and push down to secure it
7. Close the cartridge support
8. Push in the plunger
9. Squeeze the trigger and handle and see the material mixing in the tip



HOW TO REMOVE THE CARTRIDGE:

1. Press and hold the release button and pull out the plunger completely (do not remove completely from the gun but until it is fully extended)
2. Lift up the cartridge support and remove the cartridge