



## **DIRECTIONS FOR REACTIVATING HEARING AID SAVER**

### **DESICCANT:**

1. Before retiring for the night, clean your hearing aids (and ear molds if you have them) with AudioWipes™ cleaning wipes
2. Open battery doors
3. Place the hearing aids inside the jar on top of the desiccant container screen side up
4. Replace the lid and close tightly

**NOTE:** The Hearing Aid Saver uses silica gel to absorb moisture. Periodically you must reactivate this material. Desiccant beads are ready to be used when blue/purple in color; when all the desiccant beads turn pink, it is time to reactivate

### **TO REACTIVATE IN MICROWAVE OVEN:**

Remove desiccant pod from container. Place desiccant, screen side up, in microwave and heat for 30 seconds on high. Repeat if the beads have not returned to original blue/purple color. Allow to cool for 20 minutes.

### **TO REACTIVATE IN CONVENTIONAL OVEN:**

Remove desiccant pod from container and bake, screen side up, for 1 hour at 200 degrees, or until beads return to blue/purple color. Repeat if the beads have not returned to original blue/purple color. Allow to cool for 20 minutes.