



POCKETALKER® Ultra

Personal Amplifier

QUICK SETUP GUIDE

How it works


The Pocketalker Ultra amplifies sounds closest to the listener while reducing background noise.

Important: If you wear a pacemaker or similar medical device, ask your doctor or the maker of your device if you can safely use the Pocketalker.

Getting started:

1. Turn the Pocketalker on by turning the volume control counter clockwise. The **RED** indicator light on the top will turn on.
2. Turn the volume **down** before putting on the headphones.
3. Once the headphones are in place, slowly turn the volume up to a comfortable level. Keep the volume at the lowest level needed to hear.
4. If you hear feedback (a howl or squeal), turn down the volume and move the microphone away from the earphone (or headphones).
5. Adjust the tone according to your preference.
6. Hold the microphone as close to the speaker as you can.
7. When sitting down, place the Pocketalker on a table or other flat surface for best sound.

NOTE: Pocketalker is not a toy—do not allow children to play with it.

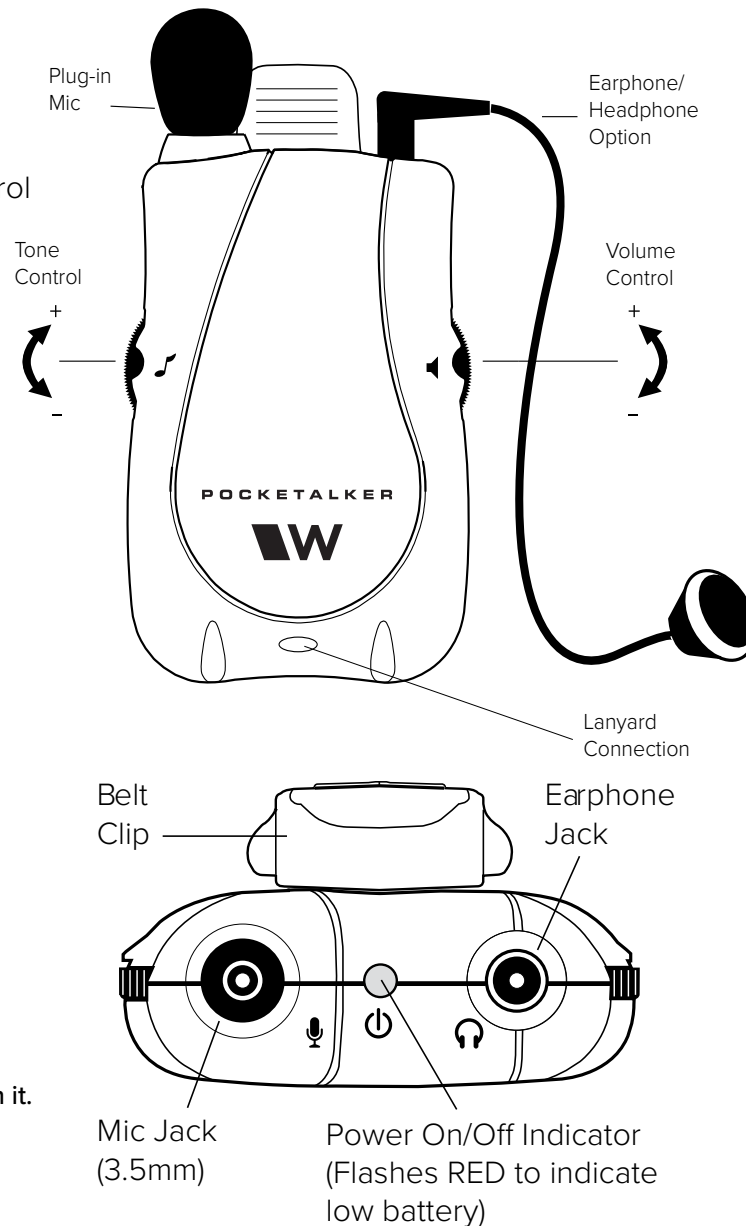

WILLIAMS AV

10300 Valley View Rd • Eden Prairie, MN 55344
800-328-6190 / 952-943-2252 • FAX: 952-943-2174
www.williamsav.com